




STUART PROSTHETIC
DENTISTRY

POLIDENT ANTIBACTERIAL DENTURE CLEANER DIRECTIONS FOR DAILY USE

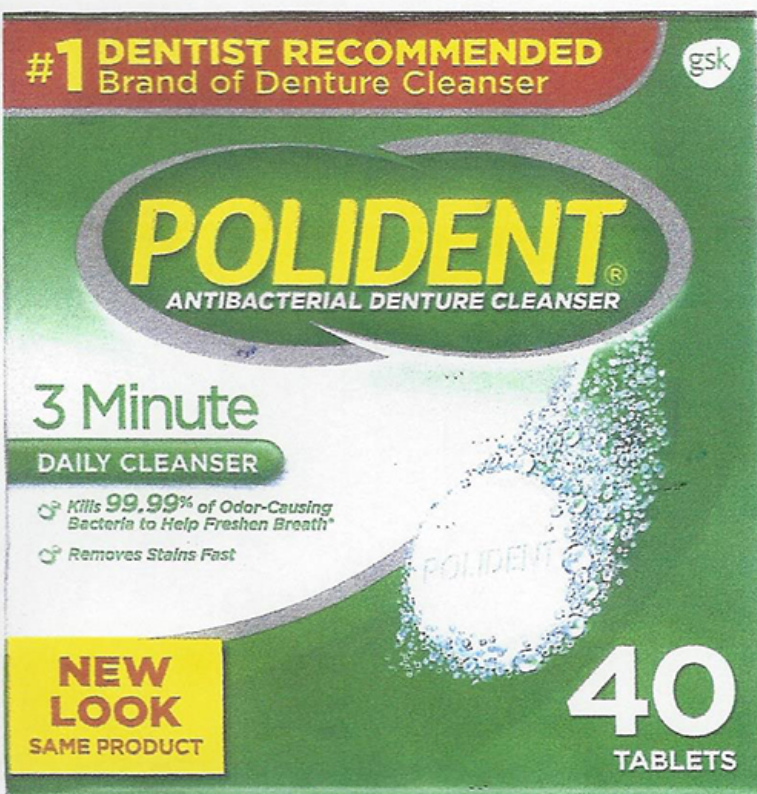

DIRECTIONS FOR DAILY USE



1. Drop one POLIDENT® cleanser tablet into enough very warm (not hot) water to cover denture or partial.
2. After soaking for 3-5 minutes, for best results, brush denture with solution using a soft brush. Discard remaining solution immediately after use.
3. Rinse denture thoroughly with running water.

CAUTION:



Keep tablets out of reach of children and those at risk of accidentally swallowing the tablet or solution



Do not place tablets or solution in mouth



Do not drink cleansing solution or use it as mouthwash



Rinse well

- Do not use if you are sensitive to any of the ingredients.
- May produce allergic reaction. If irritation occurs, discontinue use and get medical advice and/or attention.
- IF SWALLOWED: Call a Poison Control Center or doctor.
- Do not touch mouth or eyes after handling tablets or solution.
- Wash hands thoroughly after handling tablet or solution.
- Causes serious eye irritation. IF IN EYES: rinse cautiously with water for several minutes. If eye irritation persists, get medical advice and/or attention.

INGREDIENTS: sodium bicarbonate, citric acid, potassium monopersulfate, sodium carbonate, sodium carbonate peroxide, TAED, sodium benzoate, PEG-180, sodium lauryl sulfoacetate, VP/VA copolymer, flavor, blue 1 lake, blue 2, yellow 5 lake, yellow 5

Questions or comments? call toll-free 1-866-844-2796.

Polident kills odor-causing bacteria that brushing alone can miss*

Do not store above 25° C (77° F)

*soaking only, in laboratory tests

1001 SE Ocean Blvd, Suite 102
Stuart, FL 34996
(772) 286-1606